

Now available with reflective backing!  
Try our new Reflective DryTips®

Video and instructions for use available  
at [www.microbrush.com/drytips](http://www.microbrush.com/drytips)

# Ultimate Saliva Control

Far outlasts cotton rolls and other absorbent pads, and doesn't lose moisture when handled.

The DryTips® advantage  
over traditional cotton rolls:

- Complete coverage of parotid orifice and entire buccal mucosa with polyethylene film for cutting protection
- Facilitates work far back in the oral cavity
- Flexible; accommodates to cheek movements
- Comfortably stays in place, yet is easily removed with water spray
- Reflective coating for increased visibility and to aid in retraction



NEW! Reflective coating  
now available.

### Ideal for:

- Impressions
- Occlusal Adjusting
- Orthodontics
- Paint-on techniques
- Composites
- Cementation
- Fissure sealing
- Fluoride applications

### DryTips® Ordering Information:

ITEM NUMBER	SIZE	QUANTITY	COLOR
291542	Large	50	White
291543	Small	50	White
291726	Large	50	Silver
291727	Small	50	Silver

Order today through  
your preferred dealer.

For questions, call  
Microbrush® International  
at (866) 866-8698.

## Your Choice of 3 Great Offers!

### BUY 2

Two 4-packs of the  
Microbrush® Plus  
Series or Tube Series  
Applicators



### GET 1

One Box of New!  
Reflective DryTips  
for no charge



### BUY 1

One Refill Pack  
of Coated  
or Uncoated  
ContactPro® SmartBand  
Matrix Bands

### GET 1

One Refill Pack of the  
same ContactPro  
SmartBand Matrix Bands  
for no charge\*

\*See side 2 for product details



**MUST WRITE Promo Code:  
Q417 on Invoice**

### BUY 3

Three Boxes  
of DryTips® or  
Reflective DryTips®



### GET 1

One Box of  
Reflective DryTips  
for no charge



**ContactPro SmartBands can  
be used with any matrix ring!**

Full Name		Date of Purchase	
Street Address			
City	State	Zip	
E-Mail	Telephone	Fax	

\*To redeem this offer, send a copy of your qualifying invoice to: [freegoods@microbrush.com](mailto:freegoods@microbrush.com) or fax it to 888-771-7573. Purchases must be shown on one invoice. Offer limited to one per office and must be received no later than one month after the promotion expires. Offer good 10/1/2017-12/31/2017.

# CONTACT PRO<sup>®</sup>

MATRIX SYSTEM

See the ContactPro<sup>®</sup> matrix system in action at [microbrush.com/contactpro](http://microbrush.com/contactpro)



## Simple Technique for Outstanding Results

The reliable, consistent and easy to use system that provides solutions for Class II Restorations.

### 1 Place the Matrix Band



Select a band that matches the height of the tooth and place it with the concave edge toward the occlusal margin using the SmartTab at the top of the band.

### 2 Place the Wedge



Place a finger on the band to prevent dislodgment and firmly insert wedge to seal the gingival margin.

### 3 Place the Ring




Open the ring with your ContactPro<sup>®</sup> Ring Forceps and place it directly over the wedge, seating it as far gingivally as possible.

### 4 Voila!



Simple technique for outstanding results.

## CONTACT PRO<sup>®</sup> SmartBand

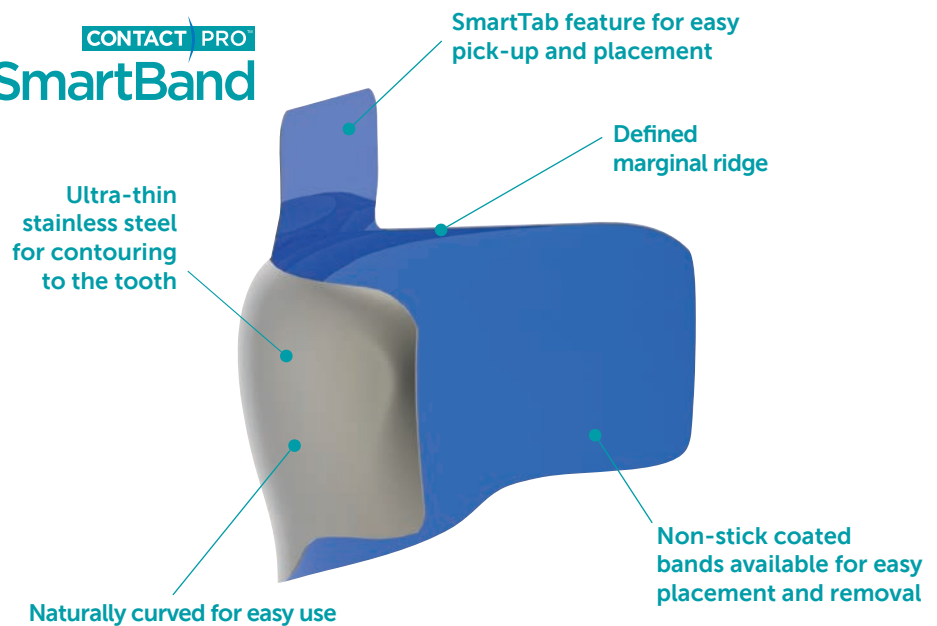

ContactPro<sup>®</sup> SmartBand Matrix Bands  
Refill Pack Ordering Information:

ITEM NUMBER	SIZE	QUANTITY	COLOR
291704	3.5mm	100ct	White
291591	4.5mm	100ct	Green
291593	5.5mm	100ct	Blue
291705	6.5mm	25ct	Purple
291706	7.5mm	25ct	Gray
291707	3.5mm	100ct	Silver
291590	4.5mm	100ct	Silver
291592	5.5mm	100ct	Silver
291708	6.5mm	25ct	Silver
291709	7.5mm	25ct	Silver

Order today through your preferred dealer.  
For questions, call Microbrush<sup>®</sup> International at (866) 866-8698.